
Instructions for use

# Family

Compressor nebulizer system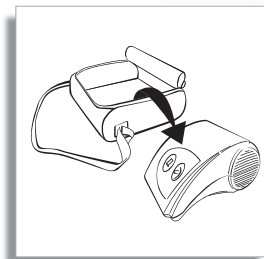

**PHILIPS**

**RESPIRONICS**

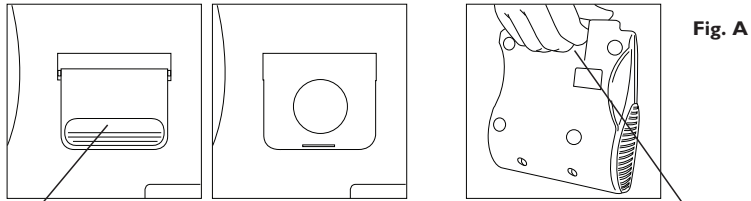
Fig. A

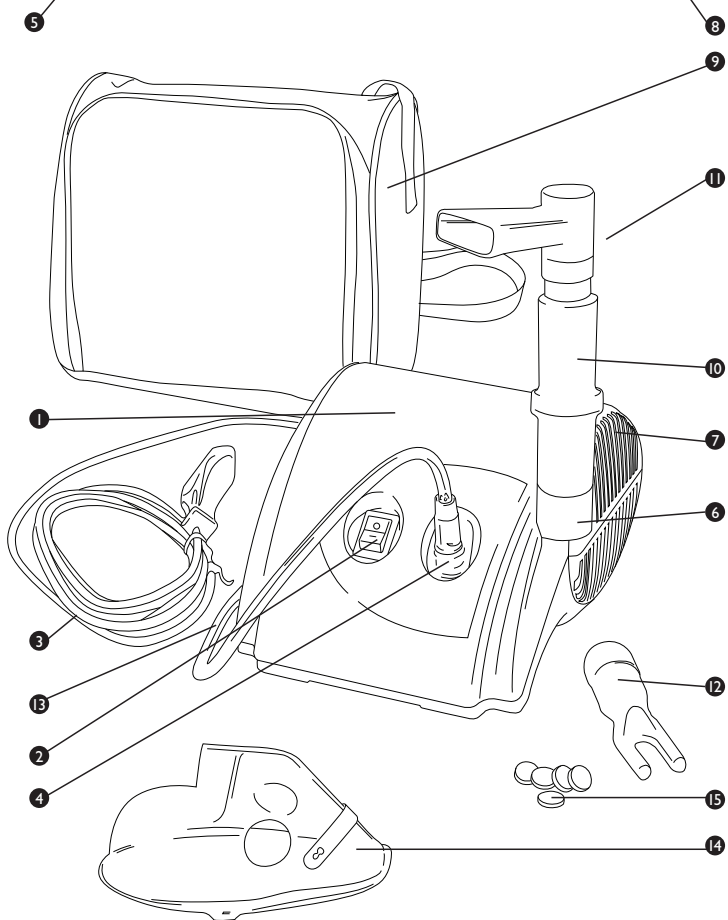
Fig. B

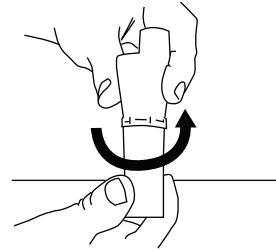
Fig. E

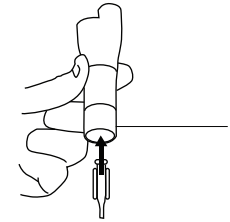
Fig. C

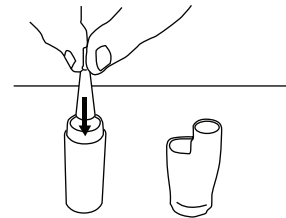
Fig. F

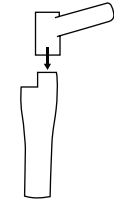
Fig. D


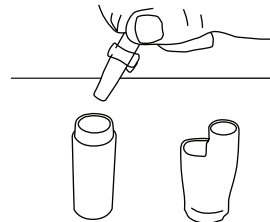
Fig. G

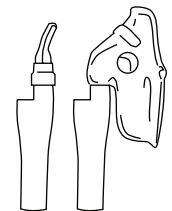
# Family

Compressor nebulizer system

## Index

EN	5
FR	11
IT	17
DE	23
PT	29
ES	35
NO	41
NL	46
PL	52
GR	58
TR	64
AR	70
ZH	76
JA	81
KO	87
RU	92
FA	98
HU	104
CS	110
BG	116
RO	122
SK	128
SL	134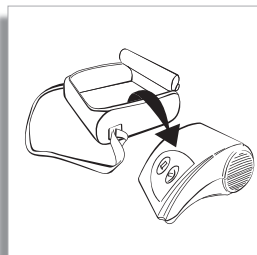

## Family components

1. Compressor housing
2. ON / OFF switch
3. Power cord
4. Air outlet
5. Filter holder
6. Nebulizer holder
7. Air vents
8. Handle

## Standard accessories

9. Carrying case (only for Silver model)
10. Jet Basic nebulizer
11. Angled mouthpiece
12. Nosepiece
13. Air tube
14. Adult mask
15. 5 spare filters

## Instructions for use

### Family

**Important:** Read these instructions carefully before using this device. If you have any questions or problems with the product, please contact your healthcare provider or Philips Respironics Customer Service at (inside the US) 1-800-962-1266, (outside the US) +1 724-387-4000.

### General information

Family is a compressor nebulizer system for aerosol therapy. Family is supplied with the Jet Basic nebulizer.

#### Caution

- 1. Only use the device for its intended purpose as described in these Instructions for use as a compressor nebulizer system for aerosol therapy. Follow the instructions of your doctor.** Any other form of use constitutes improper use and may be dangerous. The manufacturer cannot be held liable for any damage caused by improper, incorrect or unreasonable use, or if the unit is connected to electrical installations not complying with safety regulations.
2. Keep this manual for future reference.
3. Do not operate the unit in the presence of any anesthetic, inflammable mixtures or oxygen.
4. Correct device operation can be affected by electromagnetic interference that exceeds the limits indicated by the European standards in force. If this device interferes with another electrical apparatus, remove it and plug it into a different electrical outlet.
5. In case of failure and/or malfunction, read the “**Troubleshooting**” section. Do not open the compressor housing as this is not a serviceable product.
6. The proper function of the device can be compromised if original replacement parts (e.g., filters) are not used.
7. When using any electrical appliance, certain important safety measures must always be observed, including the following:
  - Use only original accessories and components;
  - Never submerge the unit in water as the device is not protected against water penetration;
  - Never touch the unit when your hands are wet or moist;
  - Do not leave the unit outdoors;
  - Place on a stable and horizontal surface when operating;
  - Make sure the air vent openings are not obstructed;
  - Do not allow unsupervised children or physically challenged individuals to use the unit;
  - Do not turn off the unit by simply pulling out the plug from the electrical outlet.
8. Make sure that the electrical rating shown on the rating plate on the bottom of the unit corresponds to your main voltage and frequency before plugging in the device.
9. Do not use any adapter, simple or multiple, and/or extension cable.

10. Do not leave the unit plugged in when not in use. Unplug the device from the electrical outlet when not in use.
11. Follow the manufacturer’s instructions for installing the device. The manufacturer cannot be held responsible for any damage caused by incorrect installation.
12. The power supply cord cannot be replaced by the user. In case the power supply cord becomes damaged, contact Philips Respironics customer service for replacement (there is no service option).
13. The power supply cord should always be fully unwound in order to prevent dangerous overheating.
14. Before performing any cleaning operation or filter replacement, disconnect the device from the main supply by unplugging it or by turning the main supply switch OFF.
15. Some parts of the unit are small enough to be swallowed by children. Do not let unattended children have access to the unit.
16. If you decide not to use the device, it is recommended to dispose of it according to the local regulations in force.
17. Remember to:
  - Use this device only with medications prescribed by your doctor;
  - Use the accessories recommended by your doctor for the therapy;
  - Use the nose piece accessory only if expressly indicated by your doctor and do not insert into nasal passage. Place close to the nasal openings and breathe normally.
18. Verify that the medications prescribed by your doctor can be used with common compressor nebulizer systems for aerosol therapy.

### Instructions for use

#### Remove the compressor from carrying case before use.

Examine the unit before the first use in order to check for possible shipping damage. When taking a treatment sit upright and relaxed.

1. After unpacking the device, check the compressor and accessories for visible damage or defects such as cracks in the plastic housing which could expose electrical components. Contact Philips Respironics customer service or your product distributor to report any damaged product.
2. **Before using the device**, proceed with the cleaning operations as described in the “**Cleaning and disinfection of the accessories**” section.
3. Open the nebulizer by turning the top counter clockwise (Fig. B).
4. Make sure the baffle is positioned correctly inside the nebulizer chamber (Fig. C).
5. Put the prescribed quantity of medication into the medicine cup (Fig. D).
6. Close the nebulizer by holding the bottom and turning the top clockwise, making sure that the unit is thoroughly closed.
7. Connect one end of the air tube to the nebulizer (Fig. E.), and the other end to the air outlet on the device (Fig. A/4).
8. Connect the angled mouthpiece (Fig. A/11) to the nebulizer. (Fig. F).

9. If you need to use the nosepiece (Fig. A/12) or the mask (Fig. A/14), take the angled mouthpiece off and connect the nosepiece or the mask directly to the nebulizer (Fig. G).
10. Plug the compressor into the electrical outlet, making sure the electrical rating shown on the rating plate on the bottom of the unit corresponds to your main voltage.
11. To start the treatment, switch the device ON by pressing the ON / OFF switch (Fig. A/2) onto the "I" position.  
**ATTENTION:** This device is rated for intermittent use: 30 minutes ON / 30 minutes OFF.
12. When the treatment has been completed, switch the unit OFF by setting the switch position to "O" and disconnect the plug from the electrical outlet.
13. Wash the nebulizer and accessories as described in the "**Cleaning and disinfection of the accessories**" section.

## Cleaning and maintenance

### Cleaning of the compressor

Use a clean dry cloth and mild detergents to clean the outside of the unit.

**ATTENTION:** During cleaning, make sure the internal parts of the unit are not in contact with liquids and the power cord is not connected.

### Cleaning and disinfection of the accessories

**After each treatment:**

The nebulizer and nosepiece can be cleaned by washing in liquid soap and water and then disinfected by boiling in water (max. 10 minutes) or in an autoclave (max. 121°C). It is suggested that the mask and the angled mouthpiece be disinfected using cold disinfecting liquids. Dry the nebulizer using a clean cloth. After drying, connect it to the air outlet (Fig. A/4) and switch the compressor ON to verify that it is functioning after cleaning.

 **Do not boil or autoclave the air tube, the mask or the angled mouthpiece.**

 **It is very important to carefully follow the cleaning and disinfecting instructions for efficient and safe product performance.**

### Replacement of the nebulizer

The nebulizer should be replaced after a long period of inactivity, or if it is obstructed by dry medication, dust, etc. With normal use, your nebulizer should last at least 6 months.

 **Use only original replacement nebulizers.**

### Replacement of the filter

The filter is to be replaced after approximately 30 hours of use, or when it appears dirty. Open the filter holder (Fig. A/5) to replace the filter (Fig. A/15) with a new one. Use original manufacturer-supplied filters only.

 **Do not use the device without the filter.**

## Troubleshooting

**The device does not turn on:**

1. Make sure the plug is fitted firmly into the electrical outlet.
2. Make sure the device has been operating within operating limits indicated in these Instructions for use (30 minutes ON / 30 minutes OFF).

**The device does not nebulize, or the nebulization is weak:**

1. Make sure the ends of the air tube are fitted tightly onto the main unit and onto the nebulizer.
2. Check whether the nebulizer is empty of medication or overfilled with medication (maximum fill is 6 ml).
3. Check whether the nebulizer nozzle is obstructed.
4. Check whether the baffle is inserted properly (Fig. C).

**If the device does not work properly after attempting these troubleshooting steps, contact an authorized distributor or Philips Respironics Customer Service at (inside the US) 1-800-962-1266, (outside the US) +1 724-387-4000.**

## Maintenance and repairs

**Never open the device.** There are no serviceable parts within the unit. The compressor requires no lubrication or maintenance.

**Technical specifications**

Model	Family
Electricity supply	230V/50Hz (#90479, 90543)
Nebulization rate	0.28 ml/min
Particle size	MMD 4.99 µm
Compressor max. pressure	230 kPa (2.30 bar)
Weight	1.55 kg
Size	(H) 167 x (L) 145 x (W) 97 mm
Noise level	55 dBA

Class II device (device with specific protection against electrical hazards).

Type B device (device with specific protection against electrical hazards).

Device should not be used around flammable substances, or in the presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide.

Device not suitable for use in anesthetic or lung ventilation systems.

**Reference to standards**

Electric safety standards EN 60601-1
Electromagnetic compatibility according to EN 60601-1-2

**Ambient conditions**

<b>Storage conditions</b>	
Temperature	MIN -25 °C – MAX 70 °C (-13 °F – 158 °F)
Humidity	MIN 10% RH – MAX 95% RH
<b>Operating conditions</b>	
Temperature	MIN 10 °C – MAX 40 °C (50 °F – 104 °F)
Humidity	MIN 10% RH – MAX 95% RH
Altitude	From 0 to 2000m above sea level
Atmospheric Pressure	700 hPa – 1060 hPa

**Device symbols**

	ON (power)		ATTENTION: Read the instructions carefully
	OFF (power)		Alternating current
	Type B		Class II
	Separate collection		

**CE**0123

**Warranty**

Respironics New Jersey, Inc. warrants the Family compressor to be free from defects in materials and workmanship under normal use and operation for a period of 3 years from date of purchase from Respironics New Jersey, Inc. The warranty is limited to repair or replacement, at Respironics New Jersey, Inc.'s sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics New Jersey, Inc. This warranty does not extend to any components or equipment subjected to misuse, improper operation, accidental damage, or unauthorized repairs, and is not extended to charges of, or for, labor repairs. All items returned must be properly packaged and shipped, prepaid, by the distributor servicing the unit. Respironics New Jersey, Inc. shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

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© 2009 Koninklijke Philips Electronics N.V. All rights reserved.
Respironics Deutschland GmbH