# B.Well

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\_\_\_\_\_

Your new ultrasonic humidifier guarantees the right humidity level in the air, and gives immediate relief to the breathing apparatus of both children and adults.

Breathing essentially aims at supplying the lungs with oxygenated air and at eliminating waste gases, like carbon dioxide.

To assure a good operation of the respiratory system it is important to remain as long as possible in healthy environments.

The humidifier will allow achieving the ideal conditions to sleep well in the night.

It is well known, that sleep is an essential mechanism to allow our brain and body to work properly. It should be a restorative antidote against daily stress, as well as an occasion to let the brain rest and increase body vigor.

To improve the quality of sleep it is important to try to relax taking a warm bath, listening to music or reading a book. Prepare the bedroom in order to make it comfortable and quiet, not too hot and with the right level of air humidity to avoid nasal congestions or obstructions.

Your humidifier with emission of cold steam is ideal for the bedroom, the room of the kids, the living-room or the office.

The ultrasound technology makes it noiseless and allows a fine and prolonged atomization, which lasts about 6-8 hours.

It is possible to adjust the steam flow according to one's needs.

Operation warning lights.

Mains operation.

MANUFACTURER: B.Well Limited  
758 Great Cambridge Road, Enfield, Middlesex, EN1 3PN. United Kingdom

**YOU ARE STRONGLY RECOMMENDED TO READ THE INSTRUCTIONS AND WARNINGS HEREIN BEFORE USING THE EQUIPMENT AND TO KEEP THE BOOKLET IN A SAFE PLACE.**

### ■ GENERAL WARNINGS

Using any electrical appliance entails compliance with a number of fundamental rules:

- Unplug the appliance from the mains straight after use and before commencing cleaning or inserting/removing accessories.
- Do not unplug by tugging on either power cord or appliance.
- If an electrical appliance falls into the water, do not attempt to retrieve it; instead, unplug immediately.
- Never immerse the appliance in water or other liquids.
- Never leave the appliance working unattended, and never allow it to be used by children, handicapped or disabled individuals.
- This appliance must be used solely for the purpose for which it was designed and in accordance with the operating instructions.

Any other use shall be considered improper and hence hazardous.

The manufacturer cannot be held responsible for any damage resulting from improper or incorrect use.

- Before using the appliance, make sure it is intact and does not feature any visible signs of damage. If in any doubt, do not use the appliance and contact your dealer.
- In the event of a fault and/or irregular operation, turn the appliance off at once without tampering with it. Always contact your dealer for repairs.
- Keep the appliance away from sources of heat.
- Make sure your hands are dry when unplugging or plugging

in the appliance and when turning the function control knob.

- This appliance must only be used with the voltage given on the bottom of the product.
- Do not leave the humidifier exposed to the weather (rain, sun, etc.), high temperatures or near computers and other electronic instruments.
- To avoid dangerous overheating, the power lead must be completely pulled out.
- Unplug the humidifier from the power socket when it is not in use.

### ■ SPECIAL WARNINGS

- Do not plug the humidifier into the power socket until the full tank is correctly positioned on the base.
- Do not use the humidifier without water; only use cold tap water without adding essences, perfumes, drugs or anything else.
- Always turn the humidifier off and unplug it from the power socket before removing the water tank from the base.
- Place the humidifier on a flat, strong and waterproof surface, keep the power lead away from sources of heat.
- Do not cover or put anything inside the openings (e.g. steam spout, the bottom of the humidifier).
- Do not touch the transducer or the water in the tank while the humidifier is running.
- **When water in the tank is completely evaporated, immediately switch off the device and disconnect the plug from the outlet. Fill the tank again and proceed according to paragraph "Instructions for use".**
- Do not pour water through the steam atomizer.

### ■ HUMIDIFIER COMPONENTS

1. Tank
2. Steam atomizer
3. Tank plug
4. Base
5. ON/OFF warning lights
6. Electric cable
7. ON/OFF switch - Min-Max steam flow regulator
8. Floating sensor
9. Transducer

### ■ OPERATING INSTRUCTIONS

- 1) Do not turn the humidifier on if the tank has not been filled with water.
- 2) Unscrew the tank plug, which is underneath the tank, by turning it in an anti-clockwise direction. Pour cold tap water into the tank (app. 1.1 litres); do not add essences, perfume, drugs or anything else to the water.
- 3) Replace the tank plug by turning it in a clockwise direction and check for water leaks. Once the tank is full, the humidifier must always be placed on a perfectly level surface. Should the water in your home be very hard, i.e. with a high content of mineral salts, use demineralized water.
- 4) Place the tank in the proper seat on the base of the humidifier. Then, connect the plug to the outlet and switch on the device rotating the switch clockwise (the yellow warning lights switch on). After

few minutes, the steam starts to come out from the suitable openings until the tank is completely empty (about 6-8 hours of operation). It is possible to adjust the intensity of the steam flow using the proper Min-Max steam flow regulator.

**5) When water has completely evaporated, the steam stops to flow out and the device switches off automatically.**

**In any case, switch off the device by rotating the switch on OFF and disconnect the plug from the outlet. Fill the tank again and restart the device as described above.**

## ■ CLEANING AND MAINTENANCE

Before any cleaning or maintenance intervention, switch off the device and disconnect the plug from the outlet.

Lift the tank from the base, unscrew the plug and empty the remaining water completely.

Rinse the tank with water. Do not use chemical or abrasive products. Carefully dry the tank and the tank plug using a soft cloth.

Do not place the humidifier base in water, just clean it with a damp cloth and dry it carefully. Take special care when you dry near the floating probe and transducer, they are very delicate and easily damaged. Never leave water in the tank when it is not in use. Once a week, we recommend cleaning the tank with a solution of water and a drop of ammonia, leave it to soak for about 20 minutes then empty and rinse well with clean water, dry all the various parts carefully with a soft cloth. To remove any limescale deposits from the transducer, use a cloth soaked in 50% water and 50% vinegar, rub it very gently and then rinse with a soft cloth soaked in water before drying very carefully. At the end of the season, ensure all the various parts of the humidifier are completely clean. Leave the tank plug loose, so that the seal is not ruined.

## ■ SPECIAL SITUATIONS

If any of the following situations occurs:

- 1) The ON/OFF indicator lights do not come on:
  - Check it is plugged into the power socket
  - Check the ON switch is on
- 2) There is no nebulising spray:
  - Check the water level
  - Check the humidifier is placed on a flat, strong surface;
  - Check there are no limescale deposits on the transducer;
  - Check the transducer is not damaged.
- 3) Condensation forms on the humidifier or the windows in the room:
  - The steam flow is too high for the size of the room or the level of humidity present.
- 4) Build up of limescale on the humidifier or walls:
  - use demineralized water.

## ■ TECHNICAL FEATURES

|                          |                        |
|--------------------------|------------------------|
| Voltage                  | 230V AC 50HZ 24W       |
| Technology:              | ultrasonic             |
| Cold steam emission:     | app. 120cc/h +/- 30 cc |
| Tank capacity:           | 1.1 litres             |
| Safety device:           | floating probe         |
| Steam emission autonomy: | 6-8 hours              |

## DISPOSAL PROCEDURE (DIR. 2002/96/EC - WEEE)

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

**⚠ Caution!** The wrong disposal of electric and electronic equipment may involve sanctions.

## ■ WARRANTY

This appliance is warranted for a period of 2 years from the date of purchase as shown by the **dealer's stamp and signature and by the purchaser's receipt for tax purposes which must be kept attached to this warranty.**

The warranty covers only any faults in manufacturing and does not cover accidental damage, incorrect use, negligence or misuse of the product. Use only the accessories provided; the use of accessories other than the approved ones can make your warranty void. Never open the appliance; in the event of any opening or tampering the warranty will automatically become invalid. The warranty does not cover either the parts subject to wear and tear or the batteries (if provided).

The warranty expires 2 years from the date of purchase; after this period any technical service shall be charged. Information regarding technical service either under the terms of this warranty or to be paid for can be obtained by contacting [info@bwell.org.uk](mailto:info@bwell.org.uk). No compensation of any kind is owed for repairs or substitutions of products under the terms of this guarantee. In the event of faults, go back to the retailer; DO NOT send the appliance directly to B.Well. Use in public places is not allowed. The manufacturer shall not be liable for any damages to people, property or pets, either directly or indirectly, caused by the non compliance with the instructions laid down in the instruction manual, in particular the warnings regarding the installation, use and maintenance of the appliance.

It is B.Well's policy to improve its products constantly; B.Well therefore reserves the right to change its products, either partially or totally, without previous notification and according to the manufacturing requirements, without either B.Well or its retailers being in any way liable for it.

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