beurer MG 230

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Items included in the package:

- Shiatsu massage mat
- Removable cover and neck pillow
- Power supply unit
- These operating instructions

Explanation of symbols

The following symbols are used in the operating instructions and on the type plate:

WARNING

Warning of risks of injury or health hazards.

CAUTION

Safety information about possible damage to appliance/accessories.

Note

Important information.

The device has double protective insulation and therefore complies with protection class 2.

Only use indoors.

1. About Shiatsu massage

Shiatsu is a form of body massage which was originally developed in Japan around 100 years ago. It is based on ideas anchored in traditional Chinese medicine, in particular the meridian system, i.e. the paths in the human body along which energy is conveyed.

The aim of any Shiatsu massage is to promote the physical, emotional and mental wellbeing of the person being treated. To do this, energy blocks and bottlenecks in the meridians need to be dissolved and the self-regulating ability of the body needs to be stimulated.

Shiatsu therapists achieve this by exercising pressure in flowing movements along the meridians. Although Shiatsu can be literally translated as “finger pressure”, practitioners also use the heels of their hands, their elbows and knees as well as their fingers. In the process, the therapist will use his or her full body weight in order to generate the required pressure.

2. Getting to know your appliance

Your Shiatsu massage device has rotating massage heads which imitate the pressure and kneading motions of a Shiatsu massage.

This product comes complete with rotating massage heads and provides a powerful and intensive deep massage for the whole back.

The device is only for **private use** and is not intended for medical or commercial purposes.

In addition, the Shiatsu massage device offers the following advantages:

- powerful Shiatsu massage along the spine with localised massage,
- rolling massage along the spine,
- adjustable massage zones (upper, lower or the entire back),
- automatic shutoff function after 15 minutes,
- with illuminated massage heads and adjustable heating function,
- removable and washable cover and neck pillow,
- separate power supply unit.

3. Proper use

This device is solely intended for massaging the backs of human beings. It cannot replace medical treatment. Do not use the massage device if any of the following warnings apply to you.

WARNING

Use of the massage device is prohibited in the following cases:

- not with changes caused by disease or injury in the back region (e.g. disc problems, open wounds),
- not on children,
- not during pregnancy,
- never while you sleep,
- not in the car,
- not on animals,
- not during activities during which an unexpected reaction may be dangerous,
- not while ingesting substances that reduce reaction times (e.g. painkiller medications, alcohol),
- never longer than 15 minutes (risk of overheating) and allow it to cool off for at least 15 minutes before using it again.

This product can only be used with a maximum weight of 110 kg (240 lbs).

Please consult your doctor if you are unsure whether or not the massage device is suitable for you.

This device is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety or they are given instructions on how to use the device.

Children should be supervised to ensure that they do not use the device as a toy.

The device has an adjustable heating function in the massage heads. Persons who do not feel heat must be careful when using this function.

You should consult your doctor before using the massage device if:

- you suffer from a serious illness or are recovering from an operation on the upper body,
- you are fitted with a cardiac pacemaker, implant or other aid,
- Thromboses,
- Diabetes,
- in case of pains of unclarified origin.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

4. Safety information

Please read these operating instructions carefully. Failure to observe the information provided below can lead to personal injury or material damage. Keep these instructions in a safe place where they can also be read by other persons using the device. Always pass on these instructions together with the appliance if you sell it or pass it on.

WARNING

Keep children away from the packaging materials. Risk of suffocation.

WARNING: Electric shock

Just like any other electrical appliance, this massage device must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- only with the supplied power supply unit at the mains voltage indicated on the power supply unit.
- Do not use the device if damage to the device itself or any of its accessories is evident.
- Never use the appliance during a thunderstorm.

In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. Never pull on the power cable or on the appliance in order to disconnect the power supply from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces. Do not pinch, kink or twist the cables. Do not stick in any pins, needles or sharp objects.

Make sure that the massage device, the switch, the power supply unit and the cable do not come into contact with water, damp or other liquids. Therefore, use the device only in dry interior rooms (e.g. never in the bath, sauna).

Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet.

Do not use the device if damage to the device itself or any of its accessories is evident.

Do not subject the device to any impact and do not drop it.

Repairs

WARNING

- Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorized dealer.
- The zip fastener on the Shiatsu massage device must not be opened. It is only fitted for reasons relating to the manufacturing of the device.
- If the power supply unit or cord is damaged, it must be replaced by an authorised dealer.

WARNING: Fire hazard

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

For this reason, the massage device must only be operated as follows:

- Never use the device under covers, such as blankets or pillows.
- Never use the device in the vicinity of petrol or other easily flammable materials.

Handling

CAUTION

Switch the device off and disconnect the power supply unit after every use and before every cleaning.

- Never insert any objects into openings on the device, and never insert anything into the rotating parts. Make sure that all moving parts can move freely at all times.
- Keep the device away from high temperatures.
- Do not stand or sit with your full weight on the moving parts of the device, or set down any objects on the device.

Disposal

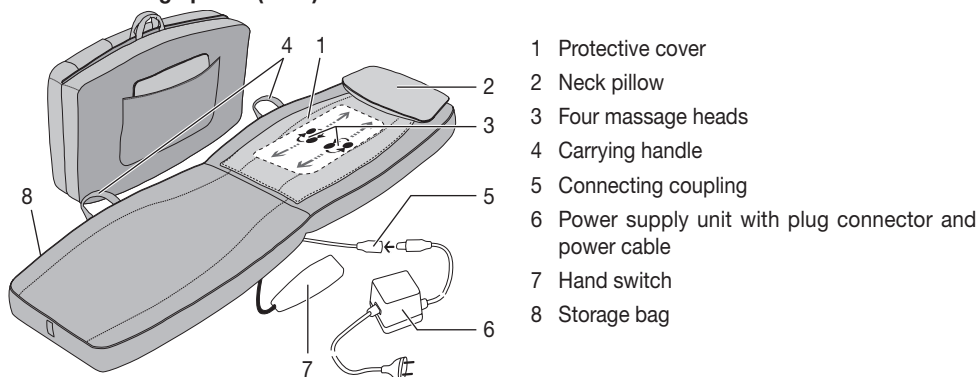
CAUTION

Follow local regulations regarding the disposal of materials. The appliance should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, contact the municipal authorities responsible for waste disposal in your area.


5. Appliance description

Shiatsu massage pillow (front)

- 1 Protective cover
- 2 Neck pillow
- 3 Four massage heads
- 4 Carrying handle
- 5 Connecting coupling
- 6 Power supply unit with plug connector and power cable
- 7 Hand switch
- 8 Storage bag

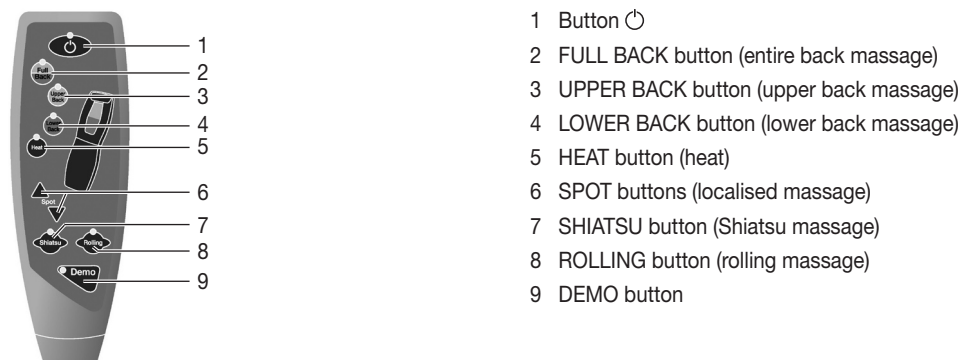Hand switch

- 1 Button
- 2 FULL BACK button (entire back massage)
- 3 UPPER BACK button (upper back massage)
- 4 LOWER BACK button (lower back massage)
- 5 HEAT button (heat)
- 6 SPOT buttons (localised massage)
- 7 SHIATSU button (Shiatsu massage)
- 8 ROLLING button (rolling massage)
- 9 DEMO button

6. Start-up

CAUTION

- **IMPORTANT:** Before using the device for the first time ensure you have released the transport lock for the massage mechanism. To do this, remove the screws using the tool provided on the rear of the massager. If the transport lock screws are not first removed and the device is used straight away, the device will become irreparably damaged and unusable. The warranty will not apply in this instance.
- Reuse the transport lock screws when transporting the device in the future. Transporting the device without using the transport lock could cause irreparable damage to the device.
- Secure the protective cover over the surface of the device using the Velcro strips. It is also possible to use the massager without the protective cover. This is generally considered to be a “harder” massage.
- Position the massager on a level surface. Ensure that the entire device is resting flat.
- Connect the power supply with the device using the connector plug.
- Connect the device to the mains by plugging the plug into the socket.

7. Operation

The massage must be pleasant and relaxing at all times. Stop the massage or change your position or the contact pressure if you find it is painful or unpleasant.

- Do not force any part of your body, in particular your fingers, between the rotating massage heads and their mountings in the device.
- Lie down comfortably on the massager. Take care to ensure you are positioned in the middle of the mat and that the massage heads to the left and right of your spine still have enough room to move.
- Switch on the massager with the  button.

The device starts automatically with the Shiatsu massage on the full back setting. The LED shows the current massage region.

You can select other settings as described below:

1. Shiatsu massage

Press the SHIATSU button. Select the desired massage region with FULL BACK (button 2), UPPER BACK (button 3) and LOWER BACK (button 4).

Spot massage, i.e. Shiatsu massage without vertical movement, can be selected by pressing one of the two SPOT buttons (number 6). The position of the localised massage can be changed by pressing and holding one of the two SPOT buttons until the desired position is reached.

2. Rolling massage

Press the ROLLING button. Select the desired massage region with FULL BACK (button 2), UPPER BACK (button 3) and LOWER BACK (button 4).

When the massage type (Shiatsu or rolling massage) is changed, the preselected region (full, upper, lower back) is retained. Note that the massage region must be selected again when switching from localised Shiatsu massage to rolling massage.

Heating function

Press the HEAT button (button 5) to switch on the heating function of the massage heads. The blue lights in the massage heads change colour to red.

- Use the massager for max. 15 minutes and then leave it to cool down. Long massages can lead to overstimulation of the muscles, creating tension instead of aiding relaxation.
You can end the massage at any time by pressing the button . Please note that the massage heads must first return to their start position when the device is switched off. This is indicated by a flashing LED light. Once the position is reached, the device switches off.
- If the device is stopped by pressing a massage area button twice, it also returns to the starting position after 15 minutes and then switches off.
- Do not disconnect the device from the mains supply during operation. Always switch it off first and wait until it has returned to the initial position.
- Disconnect the power supply unit plug from the mains after using the device.

Note

- The device has an automatic shut-off mechanism set to a maximum operating period of 15 minutes.

8. Care and storage

CLEANING

WARNING

- Before cleaning disconnect the power supply unit from the mains socket and disconnect it from the device.
- Only clean the appliance in the manner specified. Fluid must never be allowed to enter into the device or accessories.
- Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary. Do not use any cleaning agents containing solvents.

- The removable cover is machine-washable at 30 °C. Please follow the cleaning and care symbols on the label which is sewn into the cover.
- Do not use the device again until it is completely dry.

Storage

To store the device, simply fold it and then fasten the device with the Velcro fastener. Next, store the cushion in the pocket on the rear of the device.

If you do not use the device for longer periods, we recommend storing it in the original packaging in a dry place without weighting it down.

CAUTION

- If you use the screw for transport locking, take care when using the device again that it is definitely removed before switching on.

9. In the event of problems

Problem	Cause	Remedy
Message heads rotating at reduced speed.	The load on the massage heads is too great.	Take note of the maximum weight capacity.
Message heads only rotate in the upper or lower region.	The massage has been activated for the limited region.	Press the FULL BACK, UPPER BACK or LOWER BACK buttons to change the massage region.
Message heads not moving.	Appliance not connected to the power supply.	Connect the plug to the mains supply and switch on the appliance.
	Device not switched on.	Switch the device on with  . Select the massage region.
	Transport lock has not been removed, the massage mechanism is damaged.	Contact your authorized dealer or the service address.
	Overheating protection tripped, device is damaged.	Contact your authorized dealer or the service address.

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