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MANUAL BREAST PUMP

Model 1721

Thank you for purchasing a Momert product. We hope you will be satisfied with our product throughout its service life.

Please study the entire Operating Manual carefully before you start using the product. Keep the manual in a safe place for future reference. Make sure other people using the product are familiar with these instructions.

1. Introducing the Manual Breast Pump

The Momert Manual Breast Pump is a breakthrough in breast pump design. Its silicone material provides 100% reliable vacuum, whilst the silicone suction valve and the breast shield naturally imitates your baby's suckling action to provide fast milk flow – silently, comfortably and gently. The pump is easy to assemble, discreet to use. Doctors recommend that breast milk is the best nutrition for babies during the first year, combined with solid food after the first 6 months. Your milk is specially adapted to your baby's needs and contains antibodies which help protect your baby against infection and allergies.

Your Momert Manual Breast Pump will help you breastfeed longer. You can express and store your milk so that – whether you are out for a few hours, taking a well deserved rest, or back at work – your baby can still enjoy the benefits of your breast milk, even if you can't be there to give it yourself. As the pump is so silent and discreet you can take it with you anywhere, allowing you to express milk at your own convenience and maintain your milk supply.

2. Detailed parts guide Manual Breast Pump

1. Suction Dial
2. Silicon suction enclosure
3. Snap close switch
4. Handle
5. Tie Bar
6. Dustproof Cover
7. Silicon suction valve
8. Breast Shield
9. Silicone gasket ring
10. Bottle Adaptor
11. Feeding bottle cap
12. Nipple

13. Cup cover
14. Feeding Bottle
15. Base

3. Cleaning and sterilisation

Before you use your breast pump for the first time and every time before use:

Separate all parts, then wash in the dishwasher or in warm soapy water and rinse.

Sterilise in a Momert Aqua Steam Steriliser, or by boiling for 5 minutes. If using a Momert Aqua Steam Steriliser, the pump will remain sterile in the steriliser for a minimum of 3 hours, provided the lid is not removed.

Do not use abrasive or chemically aggressive substances for cleaning the unit.

4. How to assemble the Manual Breast Pump

Separate all parts and ensure pump has been cleaned and sterilised as in section 3. Wash your hands thoroughly.

- 1) Insert the tie bar (5) into the Silicon suction enclosure (2)
- 2) Place the silicone suction enclosure (2) into the Breast Shield (8). Make sure it fits securely around the rim by pressing down with your fingers to ensure a perfect seal.
- 3) Place the Suction Dial (1) on the tie bar (5) **DO NOT OVERTIGHTEN!**
- 4) Carefully insert the Silicon suction valve (7) into the breast shield (8). Make sure it fits securely.
- 5) Place the forked end of the handle (4) under the tie bar (5) and push down gently on the handle until it clicks into place.
- 6) Place the silicone gasket ring (9) into the bottom of the breast shield (8). Twist the bottle adaptor (10) on the feeding bottle (14) then the breast shield (8) onto the bottle adaptor (10). Twist gently clockwise until secure. **DO NOT OVERTIGHTEN!**
- 7) Place the breast pump on the base (15).
- 8) Put the dustproof cover (6) on the breast shield (8).

5. When to express milk

- 1) If possible, wait until your milk supply and breastfeeding schedule are well established,

normally at least 2-4 weeks after birth, unless told otherwise by your health professional or breastfeeding advisor.

2) The exceptions to the above are:

a) If you are expressing milk for your baby to be given in hospital.

b) If your breasts are engorged (painful or swollen), you can express a small amount of milk before or between feeds to ease the pain and to help your baby latch on more easily.

c) If you have sore or cracked nipples, you may wish to express your milk until they are healed.

d) If you are separated from your baby and wish to continue to breastfeed when you are reunited, you should express your milk regularly to stimulate your milk supply.

3) You will need to find the right times of the day for you to express your milk, e.g.

a) Early in the morning when your breasts are full, just before or after your baby's first feed.

b) After a feed, if your baby has not emptied both breasts.

c) Between feeds, or on a break if you have returned to work.

6. Hints to help you succeed

Using a breast pump takes practice – you may need to make several attempts before you succeed, but because the breast pump is so simple and natural to use, you will soon get used to expressing your milk.

1) Choose a time when you're not rushed and won't be interrupted.

2) Having your baby or a photograph of your baby nearby can help encourage 'let-down'.

3) Placing a warm cloth on the breasts for a few minutes before you start pumping can encourage milk flow and soothe painful breasts.

4) Warmth and relaxation can encourage milk flow. Try expressing after a bath or shower.

5) Try expressing from one breast while your baby is feeding from the other, or continue expressing just after a feed.

Practice with your pump to find the best technique for you. However, if the process becomes very painful, stop and consult your breastfeeding advisor.

7. How to use the Manual Breast Pump

1) Wash your hands thoroughly and make sure your breasts are clean. Gently squeeze a little milk from each nipple to ensure milk ducts are not blocked.

2) Make sure you have sterilised and carefully assembled your pump exactly as shown in sections 3 and 4.

3) Consult the 'Hints to help you succeed' section.

4) Relax in a comfortable chair, leaning slightly forward (use cushions to support your back). Make sure you have a glass of water nearby.

5) If the handle is in the close position, flick the switch to open it up. Press the breast shield of the pump firmly against your breast, making sure that no air can escape – otherwise there will be no suction.

6) As you gently push down on the pump handle, you will feel the suction on your breast. Select the suction strength by turning the dial on top of the breast pump. For initial shorter rapid suction motion turn the dial to low (L). For longer more powerful suction motion turn the dial to High (H). You don't need to depress the handle fully to create a vacuum – only as much as is comfortable. Your milk will soon start flowing even though you may not use all the suction the pump can generate.

7) Begin by pumping 5-6 times rapidly to initiate let-down. Then, hold the handle down for 2-3 seconds, and allow it to return to its resting place. These 2-3 second cycles imitate your baby's natural suckling pattern and allow the milk to flow into the feeding bottle between strokes.

8) Milk should begin to flow after the first few strokes. Don't worry if your milk does not flow immediately. Relax and continue pumping. Do not continue pumping for more than 5 minutes at a time if NO result is achieved. Try expressing at another time during the day. Please remember that, should the process become very painful or uncomfortable, you should stop using the pump and consult your health professional or breastfeeding advisor.

9) On average you will need to pump for 10 minutes to express 60-125ml or 2-4oz. However, this is just a guide and varies from woman to woman.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

8. Storing your breast milk

Expressed milk should be refrigerated immediately and can be stored in the refrigerator for up to 24 hours (not in the door) or in the freezer for up to 3 months.

Only milk collected with a sterile pump should be stored and fed to your baby.

If you are storing milk in the fridge to add to during the day, only add milk which has been expressed into a sterile container. Milk can only be kept like this for a maximum of 24 hours (from first expression) after which it must be used immediately or frozen for future use.

Storing your milk in the freezer

You can write the date and time of expression on the cups so that older milk can be used up first. NEVER refreeze breast milk. DO NOT add fresh breast milk to already frozen milk. ALWAYS discard any milk that is left over at the end of a feed.

9. Feeding your baby your expressed breast milk

Frozen breast milk should be defrosted overnight in the refrigerator. However, in an emergency, it can be defrosted in a jug of hot water. Once thawed use within 24 hours. Breast milk can be heated in the same way as any other bottle feed. Stand feeding bottle in a bowl of hot water, or alternatively, for a safe and fast way to warm feeds, use the Momert Bottle and Baby Food Warmer. IMPORTANT! Check milk or food temperature before feeding.

10. Troubleshooting guide

Lack of suction

Check that all the pump components have been assembled correctly

No milk being expressed

Ensure suction is created and the pump is correctly assembled. Relax and try again, expression may improve after practice. Please refer to the 'Hints to help you succeed' section.

Pain in the breast area when expressing

You may be pumping too hard. You do not need to use all the suction the pump can generate. Try depressing the handle halfway and remember the 2-3 second pumping rhythm. Consult your health professional/breastfeeding advisor.

Milk is drawn up the silicone suction valve

Remove and refit the silicone suction valve ensuring it is firmly attached and pushed far enough down the breast shield. Try leaning slightly forward.

ENVIRONMENTAL PROTECTION:

- Packaging materials and obsolete appliances should be recycled.
- The transport box may be disposed of as sorted waste.
- Polyethylene bags shall be handed over for recycling.

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Momert Co. Ltd.