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IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measurement time until the beep is heard must be maintained without exception!
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed.
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer.
- The manufacturer recommends verifying the accuracy by an authorised laboratory every 2 years.

EN

DISPOSAL

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste. For disposal of products, please contact the specialized organizations authorized to conduct disposals in accordance with the legislation of the Russian Federation.

TURNING ON THE THERMOMETER

To turn on the thermometer, press the ON/OFF button; a short beep signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «°C» appear at the upper right of the display. The thermometer is now ready for use.

USING THE THERMOMETER

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 16 seconds and that the thermometer is ready for reading. To prolong the battery life, turn off the thermometer by briefly pressing the ON/OFF button. Otherwise the thermometer will automatically turn off after about 10 minutes.

CONTRAINDICATIONS: None known STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

MEASURING METHODS

In the mouth (oral)

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/ exhaled air.

Approx. measuring time: 1 minute.

In the anus (rectal)

This is the most reliable measuring method, and is especially appropriate for infants and small children. Carefully insert the tip of the thermometer 2-3 cm into the anal aperture. Approx. measuring time: 1 minute.

In the armpit (axillary)

To receive more reliable results we recommend measuring temperature orally or rectally. A minimum measurement time of 3-5 minutes is recommended regardless of the beep sound.

CLEANING AND DISINFECTING

Name: Isopropyl alcohol 70 %.

Use an alcohol swab or cotton tissue moistened with alcohol (70% Isopropyl) to clean the thermometer. Avoid wiping the display with Isopropyl alcohol to protect it from fading. Please never immerse into liquids!

BATTERY REPLACEMENT

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and needs replacing. For the battery replacement please remove the compartment lid of the thermometer. Replace the exhausted battery by positioning the + at the top. Make sure a battery of the same type is on hand. Batteries can be purchased at any electrical goods shops. For battery disposal please refer to «safety instructions»

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This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

GUARANTEE

We grant you 5 year guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. The battery and packaging are also excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:

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