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Mi,
Momert Zrt.

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Dunaújváros, 2009. November 3.

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WARRANTY:

We give a guarantee for this product for 12 months. In case of defect of product, to get your reclamation inspected and solved, please contact the commercial unit, where the product was bought by you, with this instruction manual booklet (and the guarantee card included).

You may also contact directly the manufacturer with your reclamation. In this case the expenses of sending back the product to the manufacturer will be charged to you.

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Name: Momert

Model: 6425

Description: Electronic baby and child scale

Is in conformity with the following standards:

EMC(Electromagnetic Compatibility:

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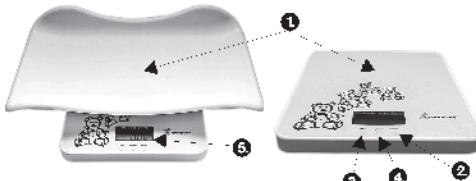
OPERATING MANUAL ELECTRONIC BABY AND CHILD SCALE MODEL 6425

Thank you for choosing our product. Your electronic baby scale is a sensitive measuring instrument, therefore please protect it from overload and from big dinamic loads ! Never leave your baby unattended on the tray !

Please, read the instruction for use before using the scale. Your new electronic baby scale has been designed for measuring your baby's weight and displaying his/her weight difference.

TECHNICAL SPECIFICATION:

Technical parameters	
Voltage	2 pcs 3V CR2032 lithium batteries
Weighing range	20 kg
Weighing increments	10 g
Weighing units	Kg, Lb, St
Working/Storage temperature	20-25°C/15-40°C
Display size	LCD diameter= 8,0 cm , 3,15 inch

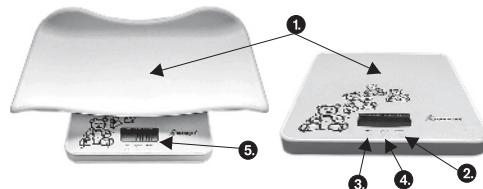
IMPORTANT SAFETY INSTRUCTIONS:

- Make sure you follow the marked polarity of battery when inserting and replacing it.
- If the appliance will be out of use for a longer time, remove the battery.
- Do not leave the unit unattended when in use.
- Do not allow children or unskilled people to handle the unit. Use the unit out of the reach of these individuals.
- Use the product only for the intended purpose specified in the Operating Manual.
- Use the unit only on a stable horizontal surface.
- Before cleaning, assembly, or disassembly of the cover, switch the unit off and remove the battery!
- Do not use abrasive or chemically aggressive substances for cleaning the unit.
- Do not use the unit if it does not operate properly, or if it has been dropped, damaged or exposed to any liquids. Take the unit to an authorized service center for testing and repair.
- The unit is suitable for domestic use only. It is not designed for commercial operation.
- Keep the unit away from heat sources, such as radiators, ovens etc. Do not expose the unit to direct sunlight or moisture.
- Do not touch the unit with wet or damp hands.
- The product is a fragile measuring device. Prevent it from crashing or falling.
- Do not repair the appliance yourself. Contact an authorized service center.

FAILURE TO FOLLOW THE MANUFACTURER'S INSTRUCTIONS MAY LEAD TO REFUSAL OF WARRANTY REPAIR!

PRODUCT DESCRIPTION:

1. Weighing surface
(baby tray)
2. ON/OFF button
3. TARE button
4. Kg/Lb/St button
5. Display

OPERATING BUTTONS:

- The scale can be switched on and off with the **ON/OFF button**. (Push the button once.)
- With the **TARE button** you can use the tare function. "0.00" appears on the display 2 seconds after pressing the button.
- **Kg, Lb, St button** is suitable to choose the required unit of measurement, the actual unit of measurement can be seen on the right side of the display.

USING THE SCALE:

1. These scales run on 2pcs of replaceable lithium batteries. Remove the protective film before use (Fig. 1).
2. Before weighing a baby, snap the tray to the measuring surface so the tray would stand stable (Fig. 2).
3. There is no need for installing the tray if you want to weight a child.
4. By pressing the ON/OFF button once "0.00" appears on the display (Fig.3).
5. Place the baby or stand the child on the scale. The measured weight is going to be displayed in a few seconds, and stored in the memory automatically. An "M" is displayed indicating the presence of data in the memory. After the next weighing the screen displays the measured weight and the difference between the measured weight and the previous weight by turns of 2 seconds. The lastly measured weight is automatically stored in the memory in case of both manual and automatic switching off.
6. By pressing the ON/OFF button once, the scale switches off. By not using the scale for 30 seconds it switches off automatically.

Fig. 1

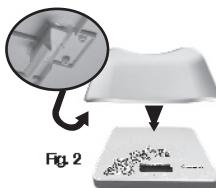
Fig. 2

Fig. 3

MEMORY FUNCTION:

After switching on the scale, an "M" is displayed which means that the last measured weight is stored in the memory, so that the weigh difference could be displayed. If you do not need the displaying of weight difference, pressing the ON/OFF button quickly twice erases the data from the memory. At this time the "M" disappears from the display. Removal of battery erases the memory too.

ABOUT TARING:

If you use not only the tray and the added weight (blanket, pillow etc.) is lighter than 5 kg, after pressing the TARE button the screen displays "0.00" and the maximal measurable weight will be 20 kg. If the added weight is heavier than 5 kg, after pressing the TARE button the screen displays "0.00" but the maximal measurable weight will be less than 20 kg (more precisely: 20 kg minus the added weight).
6.

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OTHER INFORMATION ON THE DISPLAY:

If the "Lo" warning is displayed after switching on (Fig. 4), or there is nothing on the display, the battery is dead and has to be replaced.

Fig. 4

If the sign „ERR“ appears on the display (Fig. 5.), it means that the scale is overloaded, in this case the weight must be reduced(an overload can cause sensor faults).

Fig. 5

If the message „OUT2“ appears on the display (Fig. 6.), it means sensor fault (after overloading or after a sudden, big dinamic load).

In this case, we have no warranty liability for the product.

Fig. 6

CLEANING AND MAINTENANCE

The surface of the unit may only be cleaned using a wet cloth. Do not use any detergents or hard objects as they may damage the unit's surface!

Never clean the unit under running water, do not rinse it, and do not submerge it in water!

REPAIR AND MAINTENANCE

Any extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

ENVIRONMENTAL PROTECTION:

- Packaging materials and obsolete appliances should be recycled.
- The transport box may be disposed of as sorted waste.
- Polyethylene bags shall be handed over for recycling.

Appliance recycling at the end of its service life:

A symbol on the product or its packaging indicates that this product should not go into household waste. It must be taken to the collection point of an electric and equipment recycling facility. By making sure this product is disposed of properly, you will help prevent the negative effects on the environment and human health that would otherwise result from inappropriate disposal of this product. You can learn more about recycling this product from your local authorities, a household waste disposal service or in the shop where you bought this product.

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